

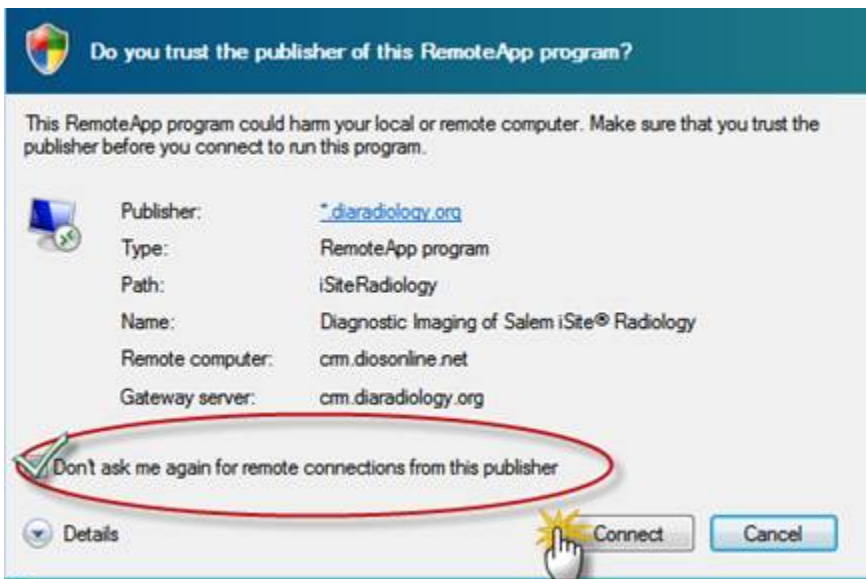


iSite Enterprise Remote App Install Guide

Download the file from the link below (Download PACS remote app) install , double-click on the install file DIAiSiteEnterprise.msi, and let it run. The install process will place an icon on your desktop labeled, “Diagnostic Imaging of Salem iSite Enterprise.” Double-click the icon to run the program. The first time you run it on each system you will be prompted for connection permissions and a Windows user name and password.

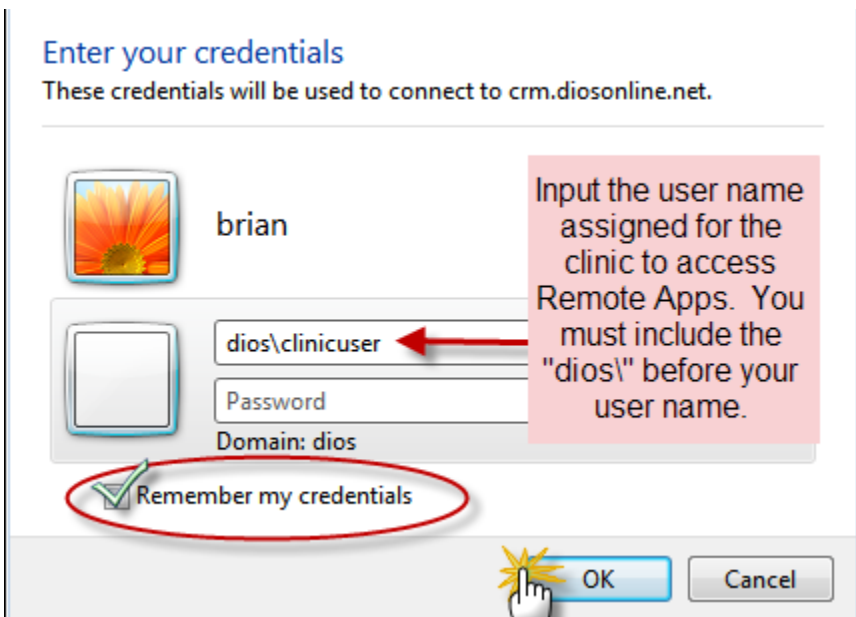

If you have Windows Vista or Windows 7, you may see a window that looks like this. Click on “Use another account” (below) If the user name is blank, proceed to the next step and input your credentials (this is different from your iSite username and password).

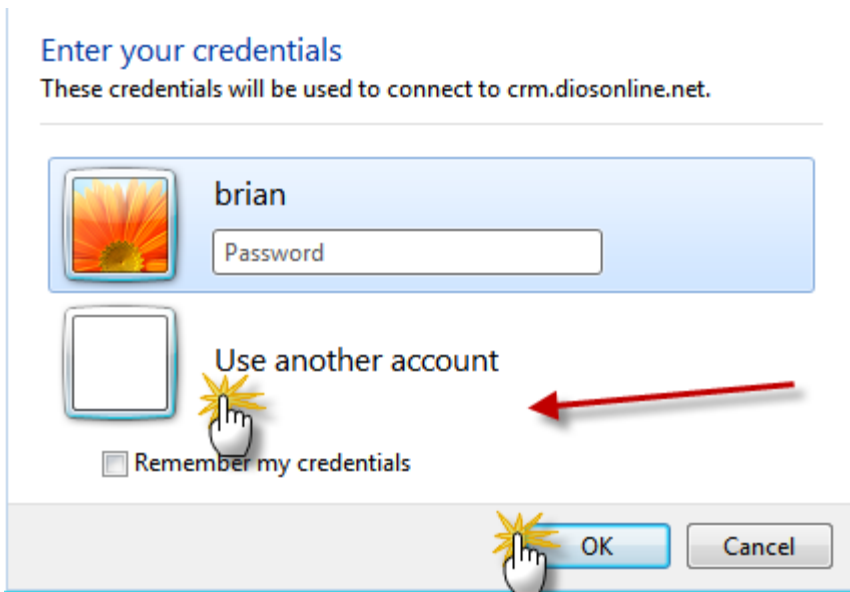

Here you will type dios\“your windows user name”. This will be the Windows user account name we assigned you (this is different from your iSite user name and password). Then type the windows password that was assigned to you. If you check the boxes to remember the information you will not have to enter it any more. After you click OK it will take a minute for the program to load (first time only).

If you have any problems installing the software please call 503-588-2674 and ask for the IT department.